**ERICSON  
LABORATOIRE**  
— PARIS —

COSMECEUTICS EXPERT SINCE 1962

# **PREMEDIKL MICRO-PICK**

**skin-jections technology**

MICRO-AIGUILLES ROTATIVES  
ROTARY MICRO-NEEDLES

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# PREMEDIKL MICRO-PICK

The penetration of active ingredients used throughout the treatment is optimized by the use of MICRO-PICK, a new medical cosmetic accessory with rotating micro-needles. The roller has 180 rotating micro-needles 0.5mm long made from stainless steel, the best quality for non-surgical use. These micro-needles create micro-canals which improve the penetration and diffusion of active ingredients in the layers of the skin. Clinical studies in Europe, the U.S.A. and South Korea have shown that the absorption of serums is increased by 1,000 due to the use of such needles. These micro-needles stimulate the synthesis of collagen and elastin and activate the growth of new cells during the following hours. Other studies have also proven that the regular use of these micro-needles improves the thickness of the skin in the medium term more than any other medico-aesthetic treatments such as intense pulsed light (IPL) or laser.

**Instructions for use:** for professional treatments, two devices should be used at the same time, one in each hand.

1. Starting from the centre of the face and moving towards the outer edge, roll the device two or three times in the cavity of each wrinkle.
2. Start by rolling at the bottom of the neck.
3. Move up 1cm and roll
4. Move up 1cm and roll
5. Start again, working upwards towards the top of the face.
6. Press more firmly on the dense and thick areas (neck and cheeks)
7. Do not press on the bonier parts of the face (forehead and cheekbones).
8. Do not use on the eyes.
9. For maximum effectiveness, the MICRO-PICK session should last approximately 10 minutes for all wrinkles. Use slow, gentle motions.
10. After each use, clean the MICRO-PICK with soapy water, rinse with clean water then apply a sterilizing product before storing it in the pouch provided.

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